


HCV Rapid Test

Diagnostic kit for antibody to hepatitis C virus
(Colloidal Gold)

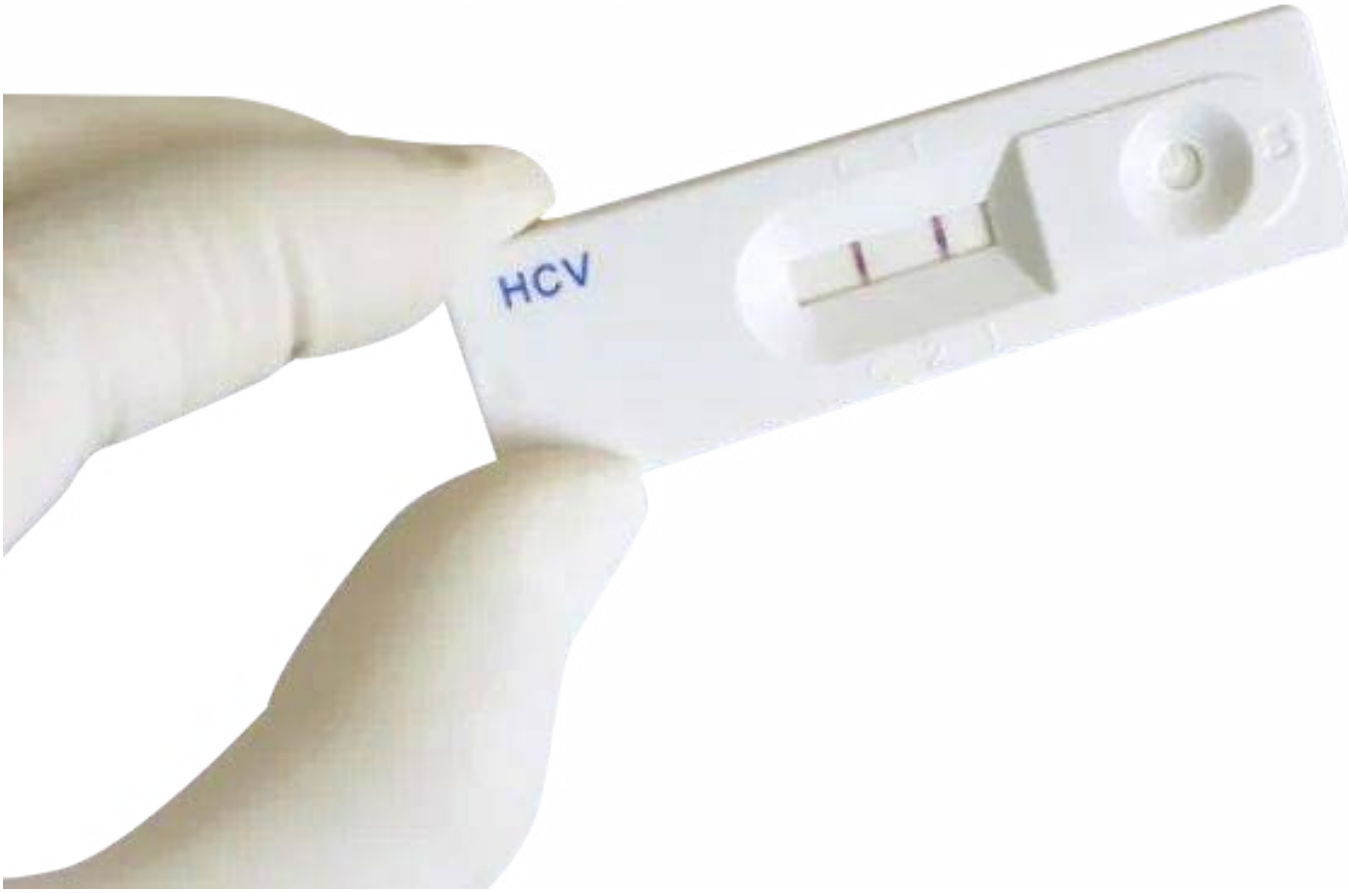

Features:

1. Easy to operation
2. Low cost
3. No extra device request
4. High stability without cool shipment
5. Low specimen request, only 50ul of Serum/Plasma/Whole blood
6. Fast detection, only 15 minutes
7. High accuracy
8. Easy to interpret results


Test Procedure:

Test Procedure:

1. Add 50ul Serum/Plasma/Whole blood into sample well
2. Add 30ul sample diluent into sample well immediately
3. Read the result between 10~15mins

Interpretation

Result valid in 15 minutes

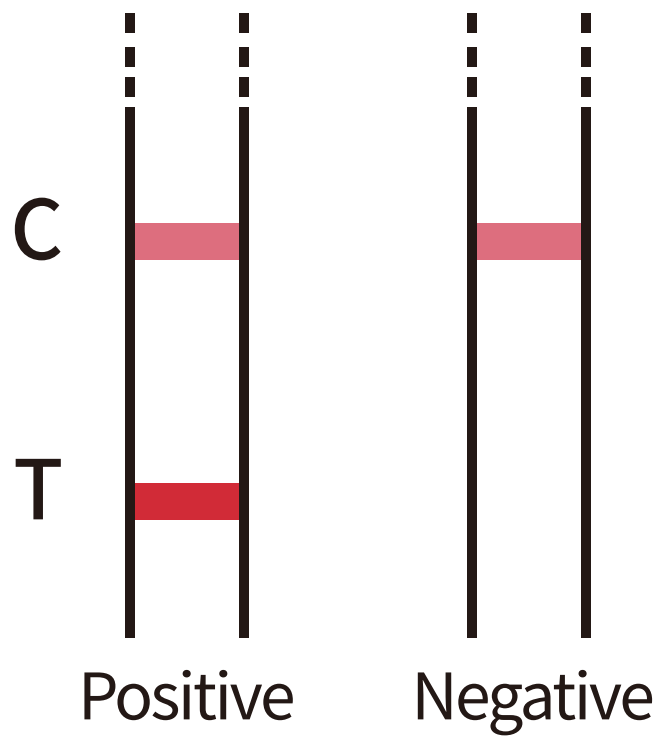

Application

The test assists the clinical detection of hepatitis C virus infection. It is suitable for hospitals, disease control centers, physical examination centers and other medical units for personnel physical examinations, preoperative testing, screening of high-risk populations, etc.